


Hemochron Signature Elite QC Testing Guide

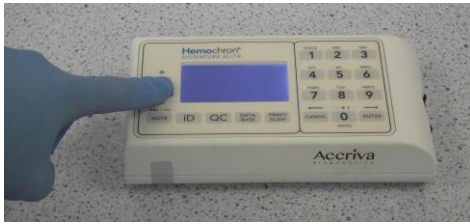


- ◆ To power on, hold 'Start'
- ◆ Remove cuvette from packaging
N.B – Ensure correct storage and handling technique is followed




- ◆ Insert cuvette into the instrument
- ◆ Instrument will bleep once cuvette is ready and on-screen countdown will commence



- ◆ Press the 'QC' button and select the correct level as displayed on-screen:
1 = Normal 2 = Abnormal

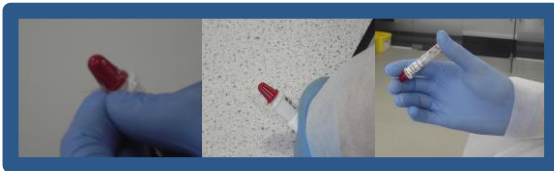

- ◆ Remove top part of plastic and place vial in protective sheath
- ◆ On a hard bench, crunch vial in several places and vigorously invert 8-10 times
N.B – Vials must be at room temperature for 1 hour prior to use

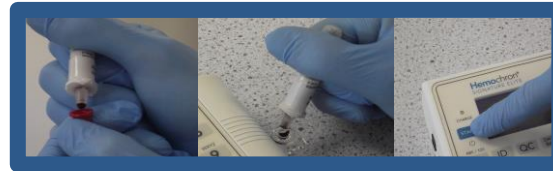


- ◆ Once reconstituted, discard the first drop into the cap
- ◆ Without delay, the second drop must then be squeezed into the sample well
- ◆ Press the 'Start' button immediately




- ◆ Once measurement is complete, check result is within the acceptable range
- ◆ If persistent failures occur, please call helpline for assistance
- ◆ Dispose of cuvette/QC as per guidelines



- ◆ With the instrument on, press 'QC'

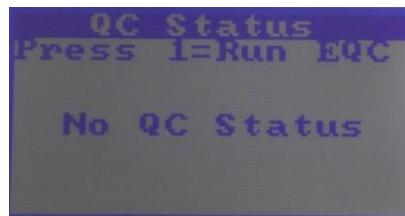

- ◆ Press '1' to run EQC
- ◆ If configured, the instrument may require entry of an operator ID

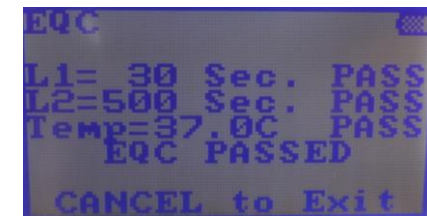

- ◆ Upon completion of EQC, results will be displayed on-screen
- ◆ If persistent failures occur, please call helpline for assistance.