

Urinalysis using the Uryxxon reader

Summary Flow Chart

Quality Control (QC) Analysis

- QC must be run at the start of the day before any patient samples
- Record all QC results (even failures) on QC sheet provided
- Consult SOP if QC fails, DO NOT USE



Preparation for Analysis

- Switch the URYXXON on or touch the screen to wake up from sleep mode
- Allow QC/Patient sample to warm up to RT for 15 minutes
- Gently invert urine to mix
- Enter your individual log in details (connected meters only)
- Enter the patient MRN in the ID field



Analysis

- Remove lid and immerse the strip so that all the tests are covered
- **Quickly** remove the stick, blot off the excess urine and place on the strip holder
- The URYXXON will detect the strip and begin the analysis countdown
- After 30 s the strip will be read internally and results printed after another 30 seconds
- Wipe strip holder before next use



Results

- The following should be written in the patient notes:
 - All results
 - Time & date of test
 - Analyser test run on
 - Person performing the test
 - Test Strip Lot Number
 - Test Strip Expiry
- Consult the SOP for follow up if needed for specific results.

For full instructions and related documents, please refer to the intranet page:

[Home](#) » [Point of Care Testing](#) » [SOPs](#)