S7770 – 2 mL × 4

989-360. 201807P.

en

cTb CALIBRATION SOLUTION S7770

Intended use: Calibration solution for the oximetry module of the ABL700 Series, ABL800 FLEX, ABL800 BASIC, ABL80 FLEX CO-OX, the ABL90 FLEX and ABL90 FLEX PLUS* analyzers.

For *in vitro* diagnostic use.

Contents: Contains salts, buffer, preservatives, and coloring agent. To get the Material Safety Data Sheet and Certificate of Traceability, contact your local Radiometer representative.

Storage and on-board stability: 24 months at 2–25 °C, with up to 14 days up to 32 °C. Refer to the label on the box or the ampoule for the expiration date. Ampoules must be stored at a constant temperature between 18 °C and 32 °C for 5 hours before use. Use the solution immediately after the ampoule is opened.

For the ABL700 Series, ABL800 FLEX and ABL800 BASIC analyzers, refer to the operator's manual for more information on the use of the cTb Calibration Solution and verification of the calibration.

NOTE: To verify an accepted cTb calibration, aspirate the S7770 solution. Use the appropriate measurement mode for your analyzer:

- ABL730, ABL720 and ABL710 analyzers: 85 µL blood sample mode
- ABL735, ABL725 and ABL715 analyzers: 195 µL blood sample mode
- ABL830, ABL820 and ABL810 analyzers: 85 µL blood sample mode
- ABL835, ABL825, ABL815 and ABL800 BASIC analyzers: 195 µL blood sample mode
- ABL837, ABL827 and ABL817 analyzers: 250 µL blood sample mode
- Check that the cTb value is within the limits shown in the table

For the ABL80 FLEX CO-OX, the ABL90 FLEX and the ABL90 FLEX PLUS* analyzers, refer to the operator's manual or the instructions for use for more information.

*ABL90 FLEX PLUS is not available in USA and Canada.

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